

SKI SHEET ^{CE}

(Part of the “SKI” range of products)

The SKI SHEET is designed for mattress evacuation, of non-ambulant patients, from places such as hospital wards and care homes in the event of a fire or other emergency. Permanently and unobtrusively fixed to the underside of the patient’s mattress, the Ski Sheet is instantly accessible when needed.

Features

- Easily accessible pockets for the storage of the cross straps.
- Elastic corner straps for attaching securely to the underside of the mattress.
- Long webbing pulling handles at each end.
- Seat belt style metal buckles and webbing cross straps.

Attaching the Ski Sheet:

- Simply lay the Ski Sheet on the base of the bed and place the mattress on the top.
- Concertina the belts and store in the sheets easy to access pockets.
- Attach the sheet to the mattress using the elastic corner loops.
- Tuck in the end pulling handles at the foot and head of the mattress.

THERE IT STAYS - READY FOR IMMEDIATE USE!

Using the Ski Sheet:

- Place pillows on the patients chest and legs for maximum patient comfort once secured.
- Pull the belts from the storage pockets (located next to the corner elastics).
- Fasten the buckles over the patient.
- Tighten the straps as tight as possible creating a U-shape.
- Slide the mattress off the bed.
- Move to a safe area (even down stairs) without the need for heavy lifting – let gravity do the work!



HA **HOSPITAL AIDS**
Evacuation Equipment Specialists

1 Kingsthorpe Business Centre, Tel: +44(0)1604 586501
63 Studland Road, Kingsthorpe, Fax: +44(0)1604 586576
Northampton, NN2 6NE. England sales@hospitalaids.co.uk
Registered in England No 1777125 www.hospitalaids.co.uk



MADE IN BRITAIN
DISTRIBUTED WORLDWIDE

Laundering Instructions

MACHINE WASHABLE.

DO NOT BLEACH.

DRIP DRY or TUMBLE DRY ON A LOW HEAT.

Washing: When laundering does become necessary the buckles on the belts should be fastened, and a temperature of up to 80°C can be used.

To protect the buckles it is recommended that the sheets should be washed in a bag.

Thorough rinsing will ensure that no detergent deposit is left.

A disinfectant may be added to the final rinse if required.

Drying: A low temperature is recommended to prevent the nylon material from becoming brittle and shrinking.



Specifications:

Packed: 27cm x 10cm x 13cm

Unrolled: 196cm x 85cm

Safety tested to: 250kg

Safe working load: 120kg – for two people

Material: Nylon

Weight: 1kg

Product codes:

SKISH01 (Standard)

SKISH02 (modified with a positioning handle situated at each pocket – As pictured)

**ONSITE DEMONSTRATIONS AND TRAINING PACKAGES ARE AVAILABLE
CONTACT US FOR MORE INFORMATION**