

# SKI SLED <sup>CE</sup>

(Part of the “SKI” range of products)


The Ski Sled has been designed for the evacuation of non-ambulant individuals in areas where space is limited, such as congested hospital wards and buildings with narrow corridors or single width fire doors.

## Features:

- The tough, slippery base enables individuals to be moved easily over virtually any interior floor surface.
- The foam pad and the internal nylon support struts give protection to the individual while being moved.
- Large foot pocket.
- 3 seatbelt style buckles (metal) and webbing straps.
- Wall mountable, wipe clean bag

## Using the SKI Sled:

- In an Emergency the Ski Sled is quickly removed from its wall mountable storage bag and taken to the individual.
- The patient is then transferred on to the Ski Sled (this can be performed either from bed/ floor/ chair).
- Place feet within the foot pocket.
- If pillows are available place these on the individuals leg and chest area (for comfort once strapped in).
- Secure the individual in place on the pad using the seat belt style cross straps.
- Using the end pulling handles the individual can be moved to a safe area. When going down stairs it is important to pull the individual feet first.

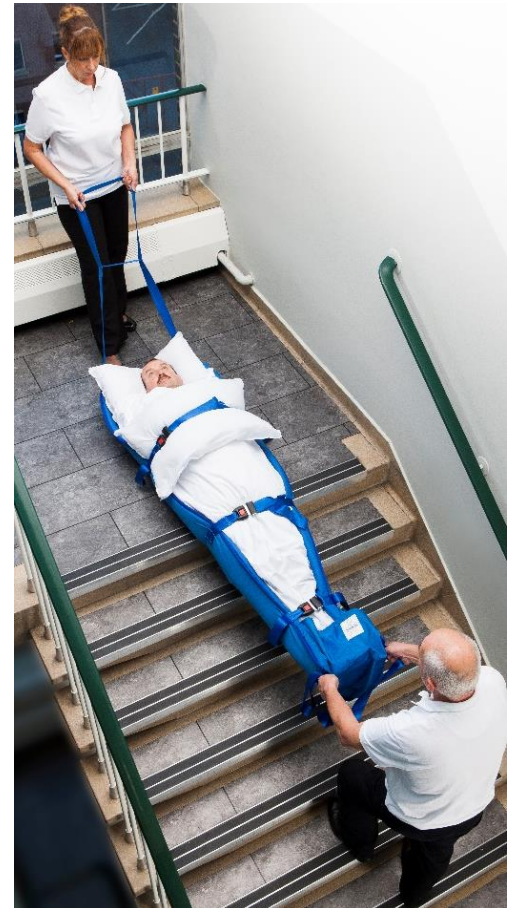

## Specifications:

Packed: 78cm x 74cm x 13cm

Unrolled: 204cm x 74cm

Padding: 3cm thick Weight: 5kg

Safety tested to: 240kg

Safe working load: 120kg – for two people

Material: Thick blue nylon

Product code: SKISL



Onsite demonstrations and training packages are available please contact us for more information



**HOSPITAL AIDS**  
Evacuation Equipment Specialists

1 Kingsthorpe Business Centre, Tel: +44(0)1604 586501  
63 Studland Road, Kingsthorpe, Fax: +44(0)1604 586576  
Northampton, NN2 6NE. England sales@hospitalaids.co.uk  
Registered in England No 1777125 www.hospitalaids.co.uk

