

# BUSCOT BABEVAC COT <sup>CE</sup>

(Part of the “SKI” range of products)

Designed for the safe evacuation of premature and new born babies from maternity wards and special care baby units in the event of fire or similar emergency.

## Features:

- Can carry up to two babies depending on size and condition.
- Secure storage for a small oxygen cylinder on the underside of the cot to allow manual ventilation of baby if required (Cylinder not included)
- Flexible strong construction allows for easy handling and adequate support and protection of babies.
- Adjustable shoulder straps.
- Wall mountable for easy, instant access.
- Easy wipe clean after use



## Using the Buscot Babevac Cot:

- Remove the Buscot from its storage bag and unfold.
- Ensure that the cot is on a flat surface
- Swaddle up the baby/ babies for warmth and place inside the cot.
- Place the straps over the shoulder - creating a single cross over the chest area.
- If manual ventilation is needed, a second carer would need to secure the cylinder to the underside of the Buscot.
- Move the baby/ babies within the Buscot, to a safe area.

## Specifications:

Packed: 62cm x 36cm x 9cm    Opened: 58cm x 30cm x 28cm

Padding: 5mm    Weight: 2kg    Material: Vinyl

Product code: **BUS001**



**HOSPITAL AIDS**  
*Evacuation Equipment Specialists*

1 Kingsthorpe Business Centre,    Tel: +44(0)1604 586501  
63 Studland Road, Kingsthorpe,    Fax: +44(0)1604 586576  
Northampton, NN2 6NE. England    sales@hospitalaids.co.uk  
Registered in England No 1777125    www.hospitalaids.co.uk



MADE IN BRITAIN  
DISTRIBUTED WORLDWIDE