

WWW.S-CAPEPLUS.COM

MANUAL WITH PICTURES

Evacuation mattress



S-CAPE KIDS
BABY & INFANT EVACUATION

Proprietary notice

The information disclosed in this manual is the property of Tetcon Global BV, Eindhoven, The Netherlands.

Tetcon BV reserves all patent rights, proprietary design rights, manufacturing rights, reproduction use rights and sales use rights thereto, and to any article disclosed therein except to the extent those rights are expressly granted to others or where not applicable to vendor proprietary parts.

Company information

Tetcon Global BV
Ruysdaelbaan 3G
5642 JJ Eindhoven
The Netherlands
www.s-capeplus.com
www.tetcon-ge.com

User information

S-CAPEKIDS Evacuation Mat comes with a comprehensive 'User Manual', for the correct usage protocol. Tetcon BV cannot be held responsible for equipment that has not been maintained in accordance with its guidelines, specification or for the incorrect use of equipment under working conditions.





A life-saving tool for a group that is fragile.

Small babies and infants are not able to escape independently during an emergency evacuation. Therefore, care workers will need an evacuation device that allows them to evacuate multiple infants or babies simultaneously, without losing precious time.

Evacuating 4 babies simultaneously by only 1 person might seem as an impossible job.

However, S-CAPEKIDS shows that your hospital or day centre can make a significant difference in terms of safety by deploying this device. It's fast, easy, comfortable and absolute safe for the little ones. It's time to meet S-CAPEKIDS.



Step 1

The S-CAPEKIDS has four compartments. Make sure the compartments are facing upwards. You can fit one baby/toddler in each compartment.



Step 2

If needed, wrap the baby in a hypothermia blanket to provide protection against hypothermia. These hypothermia blankets can be found on the inside of the Wall Cover (storage cover).



Step 3

Open the flap of the compartment. Place the baby/toddler on the mattress.



Step 4

Place the baby in the centre of the compartment / pocket. Make sure that the head is positioned onto the head support section.



Step 5

Pass the headband through the ring on the side of the headrest and close the headband securely with the hook and loop fasteners. (Note! Never place the headband over the nose or mouth).

*It's possible to mix step 5 & 6.



Step 6

You can adjust the 'footrest' to the length of the baby. You place the footrest onto the feet.

*It's possible to mix step 5 & 6.



Step 7

Now you can close the longer body flaps to fixate the baby onto the evacuation mattress. Close the longer body flaps in a 'V-shape' to make the pocket almost sealed.



Step 8

Now close the upper body flaps cross-wise around the shoulders.



Step 9

Follow the steps mentioned above for the remaining compartments so that there are four children on the mattress.



Step 10

Horizontal evacuation can be done with one or two emergency responders. You can use the pulling straps at the head or feet end.

Step 11

In case of an evacuation over the stairs, you can do this with one or two emergency workers. The person standing at the bottom of the stairs ensures that the lower leg keeps in contact with the S-CAPEKIDS.

After this you can step-by-step down.

Always make sure to keep one hand on the banister for your own safety.



Step 12

The person who controls the pull loop at the head end, provides assistance in corners and can use the pull loop as a brake it when necessary.

Always make sure to keep one hand on the banister for your own safety.





Our solutions can make a difference when disaster strikes.

THANK YOU.

DON'T HESITATE, USE S-CAPE TO EVACUATE.